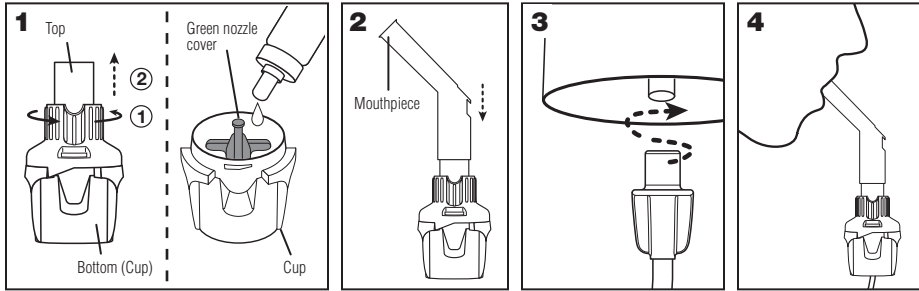
MC300[®] Reusable (R) NEBULIZER

Use Instructions

Before use, wash your hands then check the nebulizer. Replace immediately if any damage is noticed. Ensure these instructions and those supplied with the medication have been read.

1 Open the nebulizer by rotating the top counter clockwise. Pour prescribed medication into the cup. Maximum fill volume is 6 mL. Close the nebulizer, making sure the green nozzle cover is inside, then rotate the top until it stops.

2 Place the mouthpiece onto the nebulizer. If a mask is required, the nebulizer should be used with the masks listed in reorder number section. Refer to the instructions packaged with the mask.

3 Firmly attach one end of the tubing to the nebulizer. Take care not to tip the device or medication could spill. Attach the other end of the tubing to an air supply set within the following range:

Range	Flow	Pressure
Min	4 [L/min]	13 [psi]
Max (recommended)	8 [L/min]	30 [psi]

For home use, the device may be used with an **Ombra**[®] Compressor or an air supply capable of delivering the flow outlined in the table above.

When the air supply is on, ensure the device is creating a mist.

4 Place the mouthpiece in your mouth. Ensure your lips are sealed around the edge. If using a mask, position the mask lightly against the face. Inhale the aerosol breathing normally. Exhale normally through the mouthpiece (or mask). Continue treatment to sputter.

NOTE: When using the nebulizer with a mask, make sure you have a tight seal around the edges of the mask. Even tiny openings may allow outside air to enter the mask. This can reduce the amount of medication delivered. Treatment time may be longer with facemask use.

Intended Use

The nebulizer is intended to be used with pediatric (ages 2 years and above) and adult patients, who are under the care of a licensed healthcare provider or physician. The device is designed to aerosolize prescribed medication for inhalation by a patient in the hospital, clinic or home care environment. The nebulizer is a single patient use device.

Duration Of Use

This nebulizer can be used by one person for up to 6 months. Be sure to follow the use, cleaning and disinfection instructions as outlined in this insert. Replace the nebulizer within the 6 months to ensure it works correctly. We recommend you have an extra nebulizer on hand in case it becomes damaged, lost or otherwise unusable. This device is provided non-sterile.

Warnings

- **SINGLE PATIENT USE DEVICE**
- **Close supervision is required when used by or near children or anyone requiring assistance to use the device. It contains parts that may present a choking or strangulation hazard. DO NOT LEAVE CHILDREN UNATTENDED DURING TREATMENTS.**

Cautions

- Federal law restricts this device to sale by or on the order of a physician or licensed healthcare practitioner.
- For patients under the age of 5, a mask shall be used.
- Failure to properly clean the device may affect the performance.

Notes

- Always follow a healthcare professional's treatment instructions.
- Not made with BPA (Bisphenol A) or Natural Rubber Latex.

Care And Maintenance (For Home, Hospital Or Clinic)

The following section outlines nebulizer cleaning and disinfection instructions. Ensure you clean your nebulizer between each treatment using either of the methods described and disinfect at least every other treatment day.

Cleaning Instructions

Clean your nebulizer (except tubing) between each treatment.

Remove the tubing from the bottom of the device. Disassemble the nebulizer into its 4 parts (mouthpiece, green nozzle cover, top and cup).

Either of the following methods may be used:

A Hand Wash: Wash the four parts in a warm water and liquid dish soap solution. Rinse in clean warm water. Allow to air dry thoroughly before putting the device back together.

B Dishwasher (home use only): Place the four parts in a basket on the top rack. Wash in a normal wash cycle, being sure to remove the parts prior to the drying cycle. Allow parts to air dry thoroughly before putting the device back together.

NOTE:

- Do not put in the dishwasher with overly dirty dishes. Maximum of one full dishwashing cycle per day.
- Regular cleaning will help avoid build-up of residual medication. This can affect aerosol generation. It is recommended the nebulizer bottom be briefly reconnected to the air supply to clear the air pathway.

Disinfection Instructions

Always clean the nebulizer before proceeding with disinfection. The nebulizer (excluding tubing) should be disinfected at least every other treatment day. Use any one of the methods listed below.

Manufactured by:
Monaghan Medical Corporation
Plattsburgh, NY 12901 USA
customerservice@monaghanmed.com

67103-01, AWA636-01. © trademarks and registered trademarks of Monaghan Medical Corporation or an affiliate of Monaghan Medical Corporation. † trademarks and registered trademarks of their respective companies. Copyright © 2018-2020 Monaghan Medical Corporation. All rights reserved. www.monaghanmed.com

A Boiling: The four parts may be boiled in water for 5 minutes. Place the individual nebulizer parts in the water once it has reached the boiling point. Remove from water and allow to cool and fully air dry before next use.

B 70% Isopropyl Alcohol: Soak the four parts in 70% Isopropyl Alcohol for 5 minutes. Rinse well with clean water. Fully air dry before next use.

C 3% Hydrogen Peroxide: Soak the four parts in 3% Hydrogen Peroxide for 30 minutes. Rinse well with clean water. Fully air dry before next use.

D Respiratory Disinfectant: Soak the four parts in a respiratory equipment disinfectant (such as Control III[†]) following the manufacturer's instructions. Rinse well with clean water. Fully air dry before next use.

NOTE:

- Regular cleaning will help avoid build-up of residual medication. This can affect aerosol generation. It is recommended the nebulizer bottom be briefly reconnected to the air supply to clear the air pathway.

Tubing

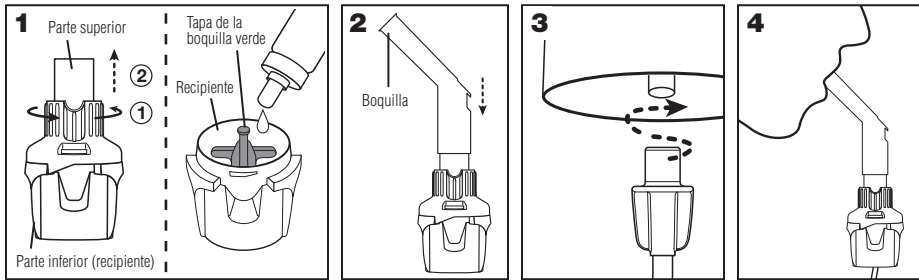
From time to time you may notice moisture build up on the inside of the tubing. This is normal. To remove the condensation, connect one end of the tubing to an air supply. Allow air to run through the tube without the nebulizer attached. This will force air through the tubing, drying the inside. To clean the exterior, wipe with a dipped tissue with either distilled water or 70% isopropyl alcohol. Ensure tubing is dry prior to storage or use.

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