Manometer Accessory

for use with the **Aerobika**® Oscillating Positive Expiratory Pressure Therapy System

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Manufactured for / Fabricado para:
 Monaghan Medical Corporation
 Plattsburgh, NY, USA 12901
T: +1-800-833-9653
F: +1-518-561-5088
 customerservice@monaghanmed.com
 www.monaghanmed.com

Manufactured by / Fabricado por:
 Trudell Medical International
 725 Baransway Drive, London, Ontario,
 Canada N5V 5G4

R_x ONLY

Qty: 1
 Cant: 1

REORDER NUMBERS

Manometer Accessory with **Aerobika**® OPEP Device **62610** Qty: **10**

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110087-001 Rev. D © Registered trade marks of Trudell Medical International. Copyright © Trudell Medical International 2015, 2018–2020. All rights reserved.

EN

INTENDED USE

The manometer is intended to be used as a single patient use accessory to the **Aerobika**® Oscillating Positive Expiratory Pressure (OPEP) device as an indicator for patient expiratory pressure.

INSTRUCTIONS FOR USE

Please read the **Aerobika**® OPEP device Instructions for Use prior to using these instructions.

An electronic version of these Instructions for Use is available for download at www.monaghanmed.com

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EN

SETTING UP YOUR MANOMETER WITH THE **AEROBIKA**® OPEP DEVICE

Before use, ensure these instructions have been read and are kept available at all times. If you have questions about the performance or usability of this product, please contact your healthcare professional prior to use.

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
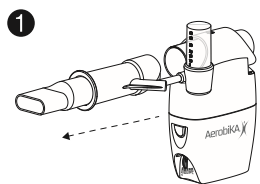
EN

CLEANING AND DISINFECTING YOUR MANOMETER ADAPTER

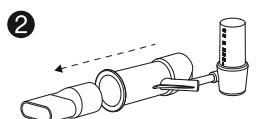
1 Remove the manometer adapter from the **Aerobika®** OPEP device.

2 Remove the mouthpiece, from the manometer adapter.

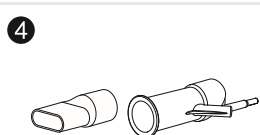3 Remove the manometer gauge from manometer adapter.

4 The mouthpiece and manometer adapter may be cleaned in soap and water, allowing parts to soak for 15 minutes. Agitate gently. Rinse parts in clean water.

OR
Place parts in a dishwasher basket on the top rack. Wash in a normal cycle. Dishwashing with overly dirty dishes is not recommended.

Shake to remove excess water and allow to air dry thoroughly before reassembly.

DISINFECTION

The mouthpiece and manometer adapter may be disinfected weekly using any of the following methods:

- A - Boil for 5 minutes
- B - Microwave Steam Bag (according to manufacturer's instructions)
- C - Soak in 70% isopropyl alcohol for 5 minutes. Rinse well.
- D - Soak in 3% hydrogen peroxide for 30 minutes. Rinse well.

Allow to air dry thoroughly before reassembly.

CAUTION
The manometer gauge cannot be cleaned or disinfected. It is a sealed unit.

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EN

CAUTIONS

- Do not leave the manometer unattended with children. It is not a toy.
- Do not soak, rinse, or sterilize the manometer gauge as these procedures may leave harmful residues or cause malfunction or inaccuracy of the device. The design and material used are not compatible with conventional cleaning and sterilization procedures.
- The manometer gauge color coding scale is only for visual support. The correct patient expiratory pressure must be determined by a medical professional.
- Make sure the manometer gauge is firmly attached to the adapter. Ensure the adapter is firmly attached to the **Aerobika®** OPEP device to avoid leaking.
- The manometer should not be used with any other device or on its own.
- Federal (USA) law restricts the sale of this device on or by the order of a physician.

DISPOSAL

The manometer (adapter and gauge) can be disposed of with domestic waste unless this is prohibited by the disposal regulations prevailing in your State.

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NOTES

- SINGLE PATIENT USE.
- Product should be replaced after 6 months of use, upon expiry, or immediately if damaged.
- This product has not been made (or manufactured) with BPA (Bisphenol A) or natural rubber latex.
- Do not share this medical device.
- Store the manometer with your **Aerobika®** OPEP device in a dry, dust-free place avoiding continuous exposure to sunlight.

TECHNICAL INFORMATION

Flow Values for Color Coded Scale Ranges	0 – 20 cm H ₂ O is the 'green' zone 21 – 40 cm H ₂ O is the 'yellow' zone 41 – 60 cm H ₂ O is the 'red' zone
Storage Conditions	-40° to 140°F (-40° to 60°C), ambient to 75% Relative Humidity (RH)
Operating Conditions	+59° to 113°F (+15° to 45°C), ambient to 85% RH
Measurement Error	0 – 30 cm H ₂ O = ± 2 cm H ₂ O 30 – 40 cm H ₂ O = ± 3 cm H ₂ O 40 – 60 cm H ₂ O = ± 5 cm H ₂ O

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