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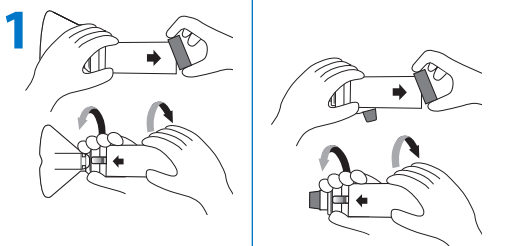
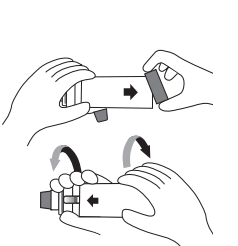
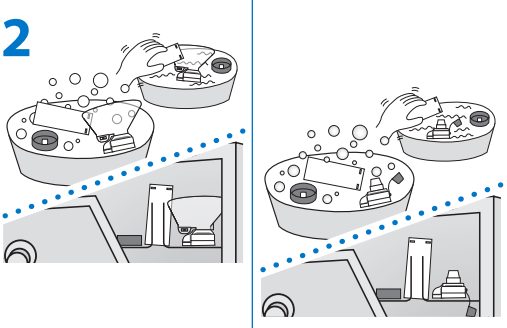
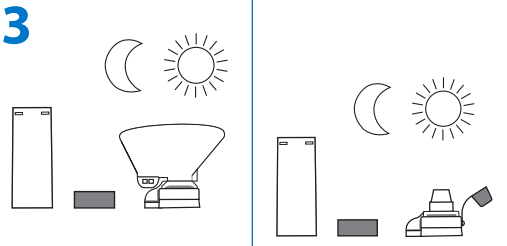
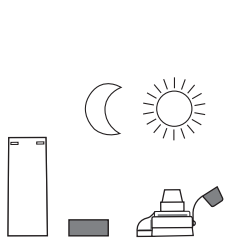
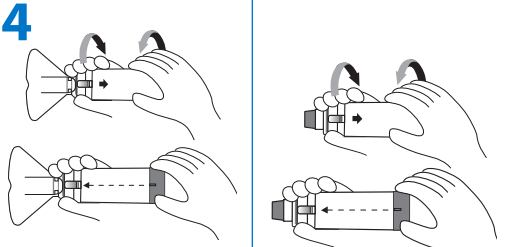
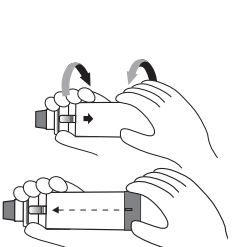
Notes

- SINGLE PATIENT USE
- The device is provided non-sterile and should not be boiled or sterilized.
- Storage and operating range 41°F–104°F (5°C–40°C) at 15 to 95% relative humidity.
- Product should be replaced after 12 months of use.
- Do not share this medical device.
- If you notice medication build-up in your chamber, wash the inside of the chamber gently with a soft cloth.
- Dishwashing with overly dirty dishes is not recommended.
- Dishwasher validated up to 158°F (70°C).
- If cleaning in a dishwasher use a rinse aid.

⚠ Cautions

- **Do not leave the chamber unattended with children. This is not a toy.**
- **Product may be permanently damaged if boiled, sterilized or cleaned in a dishwasher at a temperature above 158°F (70°C).**
- **Federal (USA) law restricts the sale of this device on or by the order of a physician.**

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Manufactured by / Fabricado por:

monaghan

Monaghan Medical Corporation.
153 Industrial Blvd., Plattsburgh, NY 12901
www.monaghanmed.com

INDICATIONS FOR USE

This product is intended to be used by adult and pediatric patients who are under the care or treatment of a physician or licensed healthcare professional. The device is intended to be used by these patients to administer aerosolized medication from most pressurized Metered Dose Inhalers and Soft Mist Inhalers. The intended environments for use include the home, hospitals and clinics. It is a single patient, multiple use device.

BEFORE EACH USE

This chamber can be used directly out-of-package. Ensure these instructions and the instructions supplied with the inhaler have been read and are kept available at all times. Carefully examine the chamber and replace immediately if it is damaged or has missing parts.

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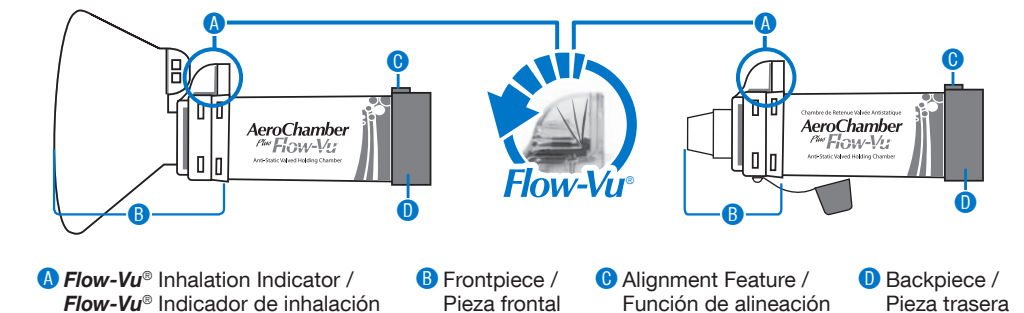
FIGURE / FIGURA 1


This device has not been made (or manufactured) with BPA (Bisphenol A) or Natural Rubber Latex.

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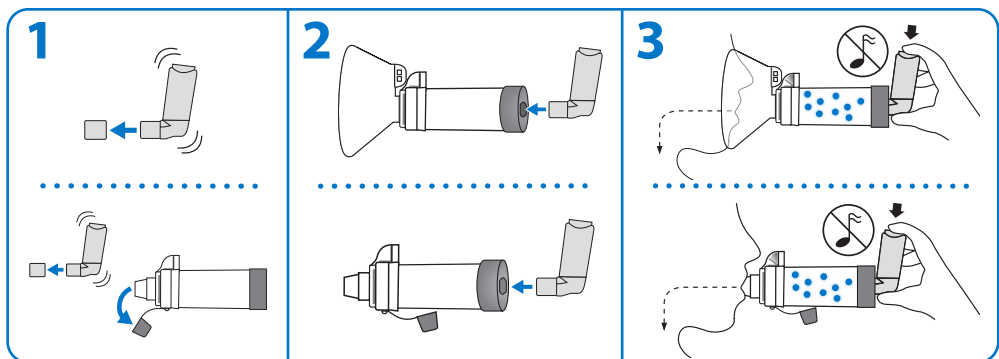HOW TO USE YOUR CHAMBER WITH METERED DOSE INHALERS (PUFFERS)

For preparation of the inhaler, follow the instructions supplied with the inhaler.

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2 MOUTHPIECE: Insert the inhaler into the backpiece of the chamber. Put mouthpiece into mouth and close your lips around it to ensure an effective seal.

Notes

- Inhalation — SLOW DOWN if you hear the whistle sound, it means you are inhaling too quickly (on adult models).
- If someone is helping you they can use the **Flow-Vu®** Inhalation Indicator to ensure a good seal, coordinate pressing the inhaler with inhalation and to count the number of breaths taken. The **Flow-Vu®** Indicator moves toward you as you inhale and only moves if you have a proper seal.
- Administer one (1) puff at a time. Follow instructions supplied with the inhaler on how long to wait before repeating.

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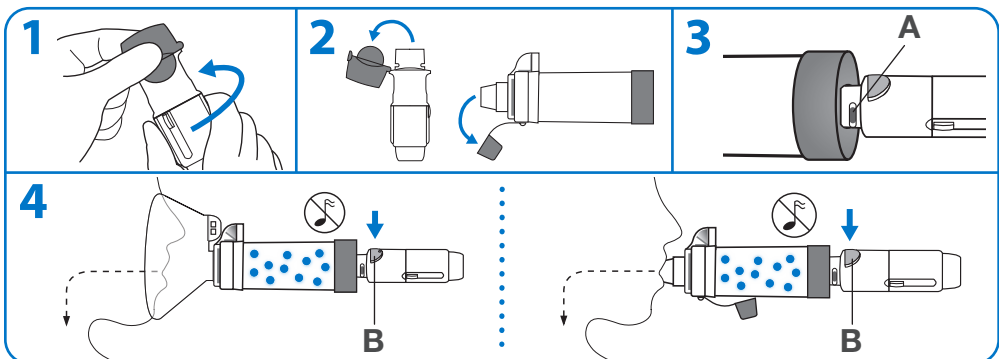
HOW TO USE YOUR CHAMBER WITH RESPIMAT+ SOFT MIST INHALERS

For preparation of the inhaler, follow the instructions supplied with the inhaler.

- 1** Hold the inhaler upright and turn the clear base in the direction of the arrows on the label until it clicks. (Keep the inhaler cap closed).
- 2** Open the cap of the inhaler until it snaps fully open. Remove the cap from the chamber.
- 3** Insert inhaler into the backpiece of the chamber. **Do not cover the air vents (A)** on the inhaler.
- 2 MOUTHPIECE:** Place mouthpiece into mouth and close lips around it to ensure an effective seal.
- 4 MASK:** Breathe out gently and press the **dose release button (B)** on the inhaler at the beginning of a slow inhalation. Maintain seal for 5 – 6 breaths after the inhaler button is pressed.
- 4 MOUTHPIECE:** Exhale then press the **dose release button (B)** on the inhaler at the beginning of inhalation. Inhale slowly and deeply through the chamber until a full breath has been taken. Hold your breath for 5-10 seconds before exhaling. **OR** Exhale then press the **dose release button (B)** on the inhaler at the beginning of a slow inhalation. Breathe in and out through the chamber for 2-3 breaths keeping lips sealed around chamber mouthpiece.

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CLEANING INSTRUCTIONS FOR HOME, HOSPITAL OR CLINIC

This chamber can be used right out-of-package and then cleaned weekly.

CHAMBER WITH MASK	CHAMBER WITH MOUTHPIECE	
		Remove the backpiece (Figure 1, D). To detach the frontpiece (Figure 1, B), twist chamber as shown.
		Soak the parts for 15 minutes in a mild solution of liquid dish detergent and lukewarm clean water. Agitate gently. Rinse parts in clean water OR place parts in top rack of dishwasher. Ensure product is securely placed face up as shown. Run the dishwasher on a normal or light cycle. Do not heat dry.
		Shake out excess water and allow to air dry in a vertical position. Ensure parts are dry before reassembly.
		To reassemble, fit the frontpiece on the end of the chamber and twist firmly until securely locked into position. Center the alignment feature on the backpiece with the Flow-Vu® Inhalation Indicator, as shown. Press firmly to attach the backpiece.

Signature Manifest

Document Number: MMC-ART-0371

Revision: 00

Title: Product Insert, AC Plus Flow-Vu aVHC

Effective Date: 07 Sep 2023

All dates and times are in Canada/Eastern.

Document Packet_CC-0186,pkt 2 of 2-Artw

4: Marketing Approval

Name/Signature	Title	Date	Meaning/Reason
Chris Patrick (CPATRICK)	Marketing Manager - U.S	05 Sep 2023, 04:11:45 PM	Approved

5: Regulatory Approval

Name/Signature	Title	Date	Meaning/Reason
Janice Breen (JBREEN)	Regulatory Affairs Associate	05 Sep 2023, 07:44:49 PM	Approved